

Fast Set Alginate - Mint

Dust-free alginates for impressions, complying with: ISO 21563 and is for dental use only.

Indications for use:

Creamy consistency alginate, does not compress tissue, especially suitable for preliminary edentulous jaw impressions. Flavour: Mint.

Dosing and mixing:

- Open the bag and pour the alginate into an airtight container. Keep the lot number and expiry date of the product by cutting the relevant details off the bag. Shake 2 or 3 times to disperse the powder then proceed with the mixing. Using the dosing spoon, remove the powder without compressing it, levelling off or removing any excess alginate by passing a flat spatula over the top of the spoon.
- Pour the powder into a rubber cup. Repeat the operation until the required quantity is reached. Add 1/3 of a measure of water (18ml) for every spoonful of powder (9g). E.g. for 2 measures of powder (18g), or one impression tray, add 2/3 of a measure of water (36ml).
- Close the container and store it at 5-27°C/41-80°F.
- Add the water to the powder and mix for the times specified in the table (see below).
- Spread the material over the impression tray and insert into the mouth before the end of the working time.
- Wait for the material to harden until the end of the recommended time in mouth. In the case of coloured alginates, working time is easily recognised by the colour changes described previously.

Times for clinical use:

Mixing time	Time in mouth (min)	Impression taking	Setting time	Total working time
30"	1'10"	1'00"	2'10"	2'00"

*Times for clinical use are indexed to the start of mixing with deionised water at 23°C/73°F. Colder water retards setting, warmer water speeds up setting. Variations in water hardness may result in deviations in times. According to ISO 21563 specifications.

Warning: If susceptible may possible to cause phenomena of irritation or other allergic reactions.

Cleaning and disinfection of the impression:

After taking the impression, carefully rinse it in order to remove any traces of saliva, drain off the excess water and disinfect immediately.

Storing the impression:

If the gypsum cannot be immediately pour, drain off the excess water and store the impression in a hermetically sealed bag. Under these conditions the gypsum pouring can be deferred for up to 48 hours after the impression taking.

FOR DENTAL USE ONLY.

Presentation: Fast Set Alginate - Mint, 500g  IAA310

This product is intended to be used as advised by the manufacturer. A non-exhaustive list of product use information is supplied on the packaging and product labelling. We hereby disclaim all liability associated with any complaints or events resulting in inadequate product performance, injuries or harm that may have been caused by the method or technique of using the product if the manufacturer's recommendations were not followed.

 Caution	 Consult instructions for use	 5°C - 27°C Temperature Limit
--	--	--

